



GLUCOMEN® AREO QUICK START GUIDE

1



Introduce a new GlucoMen® areo test strip into the top of the meter.

2



Obtain a drop of blood.

3



Apply one drop of blood onto the end of the test strip.

4



After 5 seconds, the measurement result is displayed.

5



Press the eject button to remove the test strip hygienically.




GLUCOMEN® AREO DISPLAY MESSAGE

Message

What does this message mean?

What should I do?

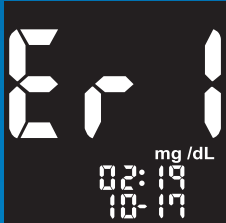

Hardware error

Contact our service team
Phone: +49 -800 - 45 82 66 36
(free)



Test strip damaged or soiled

Use a new test strip




Blood sample error

Apply more blood to a new test strip



Test strip removed too early

Repeat the test with a new test strip

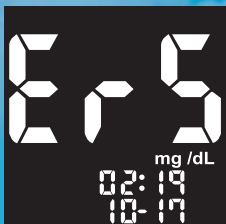

NFC connection lost

Repeat transmission