

GlucoMen® areo Quick Start Guide

How to perform a blood glucose measurement

1



Introduce a new GlucoMen® areo test strip into the top of the meter.

2



Obtain a drop of blood.

3



Apply one drop of blood onto the end of the test strip.

4



After 5 seconds, the measurement result is displayed.

5



Press the eject button to remove the test strip hygienically.



GlucoMen[®] areo Display Message

Message

What does this message mean?

What should I do?



Hardware error

Contact our service team
Phone: +49 - 800 - 45 82 66 36
(free)



Test strip damaged
or soiled

Use a new test strip



Blood sample error

Apply more blood to a new
test strip



Test strip removed
too early

Repeat the test with a
new test strip



NFC connection lost

Repeat transmission



If you have any questions or suggestions, we will be glad to be at your service.
Free service phone +49 - 800 - 45826636
(Mo - Th 08:00 - 18:00 o'clock, Fr 08:00 - 16:00 o'clock)
www.glucomenareo.de | glucomen@berlin-chemie.de

GlucoMen[®]
areo

Just measure easily